



ColoGenesis HealthCare Pvt. Ltd.
A Leading Collagen Manufacturer, India.

Presents

NanoColl

Nano Crystalline Silver in Collagen Base



Indications

NanoColl is indicated in the treatment of Burns and Wounds.

Product Description :

NanoColl is non greasy, water soluble, non-oily, clear odorless gel. It is composed of colloidal silver nano particle and highly purified type I collagen.



25°C

Storage :

Store below 25°C.
Do not freeze.

Available Sizes

- ✓ 10 g Tube
- ✓ 15 g Tube
- ✓ 25 g Tube
- ✓ 30 g Tube
- ✓ 50 g Tube

Mechanism Of Action

- Silver nano particles play a distinct role in preventing infection and decreasing bacterial load in the wound by their broad spectrum antimicrobial properties and their surface modification properties provides easy incorporation of nano silver in to collagen base to improve wound healing treatment.
- Collagen is known for its wound healing property by aiding cellular activity and providing an organized matrix in the skin.

Precautions :

- NanoColl is contraindicated in patients who have shown sensitivity to collagen or Silver particles.
- It is for external use only not for surgical implantations.

Directions For Use

- ✓ Thoroughly cleanse the wound with sterile saline solution.
- ✓ Apply the NanoColl as required to cover the entire wound.
- ✓ Cover it with appropriate secondary dressing if necessary.
- ✓ Keep the cap tightly closed after use.

Composition:

Nano Crystalline Silver 0.002% Gel in Collagen Base (Q.S.)

Application Of NanoColl

Day 3



Day 14

